


### Nasal Masks



**F&P Eson 2**

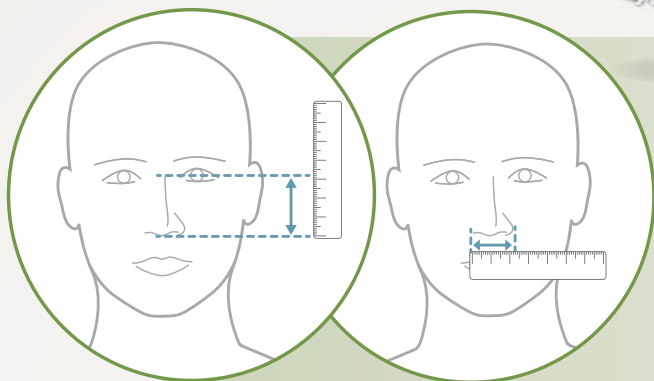

Nose Height

Nose Width

**Small**  
**Medium**  
**Large**


**Nose height**

Less than **4.4 cm (1.73 inches)**  
Between **4.4 cm and 5.2 cm (1.73 inches and 2.05 inches)**  
Greater than **5.2 cm (2.95 inches)**

**Nose width**

Less than **3.7 cm (1.46 inches)**  
Between **3.7 cm and 4.1 cm (1.46 inches and 1.61 inches)**  
Greater than **4.1 cm (1.61 inches)**

### Full Face Masks

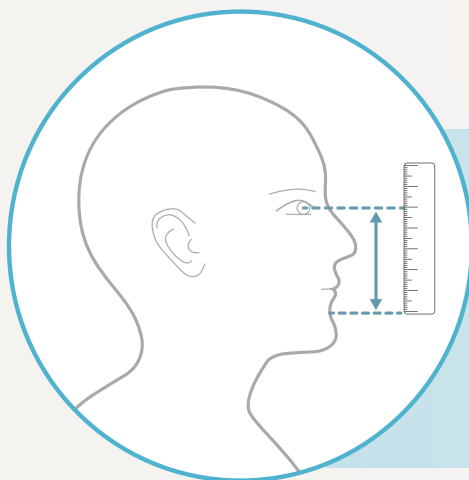

**F&P Vitera**



**F&P Simplus**

**Small** Less than **8.9 cm (3.5 inches)**  
**Medium** Between **8.9 cm and 10 cm (3.5 inches and 3.94 inches)**  
**Large** Greater than **10 cm (3.94 inches)**

**Small** Less than **6.6 cm (2.59 inches)**  
**Medium** Between **6.6 cm and 7.5 cm (2.59 inches and 2.95 inches)**  
**Large** Greater than **7.5 cm (2.95 inches)**

Measuring these points for sizing in a mirror may make the process easier for you.

These products may not be right for you. Read the warnings and instructions for use before purchase.

For further instructions on sizing, please refer to the Eson™ 2 and Simplus™ sizing guide REF 185048403 or Vitera™ sizing guide REF 612027

F&P, Eson, Vitera and Simplus are trademarks of Fisher & Paykel Healthcare Limited.  
For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

Prescription only. Use only as directed. Always follow the instructions for use.  
Your healthcare professional will advise you whether this product is suitable for you/your condition.