



CPAPMAN.COM
BEST SERVICE SINCE 1997

Click Here To Order Today

Swift FX components

Swift™ FX
Nasal Pillows Mask



PRODUCT CODES

Swift FX Nasal Pillows Mask

Complete System

Size	Part Number
Swift FX	61500 ¹
Swift FX for Her	61540 ²

Reimbursement Codes

HCPCS	Code Descriptor
A7034	Mask interface – mask or cannula, 1 per 3 months
A7033	Pillow for use on nasal cannula type interface – replacement, 2 pairs per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

1 Includes: Small (S), Medium (M) and Large (L) pillows and blue backstrap. Extra Small (XS) pillows are available separately.

2 Includes Extra Small (XS), Small (S) and Medium (M) pillows, pink backstrap and pink soft wraps. Large (L) pillows are available separately.

⚠ For details, refer to the Swift FX or Swift FX for Her Nasal Pillows Mask User Guide.

Fitting



- Holding the pillows at your nose, pull the headgear over your head.
- Headgear should sit midway between the ears and eyes.



- Adjust the top strap first.
- Pull the strap through the buckle one notch at a time.
- Do not overtighten.



- Adjust the backstrap to fine-tune the fitting if necessary.
- Do not overtighten.
- The backstrap can be positioned over or under the hair.



- Swift FX pillows are designed to provide a natural fit.
- Minor adjustment can help position the pillows at the most comfortable angle in the nose.
- Connect air tubing.
- Turn on the airflow device.



- With the device turned on, check that the nasal pillows are properly inflated. To do this, pull the nasal pillows away from the face and cheek, make sure they are inflated and reposition them in the nose.